

FAQ

SMART XTRACT® SR

WHAT IS THE PURPOSE OF THE SMART XTRACT® SR?

- > The SMART XTRACT® SR is used to quickly evacuate a casualty from their zone of injury.
- > The flexible system allows for use with or without a hard plastic "sled", based on conditions encountered.

HOW DO I PUT A CASUALTY ONTO THE SMART XTRACT® SR? ?

- > Loading a casualty onto the SMART XTRACT® is very similar to using a simple patient transport fabric stretcher.
- > Turn the casualty onto their side and position the SMART XTRACT® under the casualty, roll back and continue with the positioning the casualty correctly.
- > Pulling on the red cord will adapt the system to the casualty's body length and individual size.

CAN THE SMART XTRACT® SR BE USED ALONE FOR VERTICAL AND HORIZONTAL LIFTING?

- > No. This current model of the SMART XTRACT® is not approved for this use.

ARE THERE ANY SPECIAL ACCESSORIES THAT CAN BE USED WITH THE SMART XTRACT® SR??

- > The system can be used with or without the sled base.
- > If purchased without the sled base the sled base is available as a separate purchasing item.
- > There is an "Element Protection System" (EPS) covering that is also available to add additional cover and protection from wind and rain.

HOW SHOULD I BEST PROTECT THE CASUALTY FROM THE COLD?

- > To maximize protection from cold environments, or to assist in warming a casualty who is hypothermic, we recommend using either a passive warming product, such as the Blizzard Reflexell™ blanket or the Mediwrap® blanket.
- > For more severe cases of hypothermia, it would be best to use an active warming product such as a Ready-Heat™ blanket.

MEDICAL SALES CONSULTANTS AUSTRIA GMBH

Wachaustrasse 13, 3123 Neustift, Austria
T: +43 2786 631 38, E-Mail: info@medicalsci.com
Web: www.medicalsci.at

MEDICAL SALES CONSULTANTS GERMANY GMBH

Vor dem Schonekindtor 13, 59494 Soest, Germany
T: +49 2921 969 78 50, E-Mail: info@medicalscg.de
Web: www.medicalscg.de